AXIOM

BACKS

CB | FB | FL | LB | DL

USER INSTRUCTION MANUAL AND WARRANTY

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2024-02-14

INTRODUCTION

Thank you for purchasing a Ki Mobility product!

WARNING

WARNING: Do not use this without first reading this entire manual. Use without first reading this entire manual could result in serious injury to the user and/or attendant.

Ki Mobility recommends training by an Assistive Technology Professional (ATP) or other competent clinical or technical professional in the safe use of this product prior to use.

If you have any questions or concerns about any aspect of this product, this manual or the service provided, do not hesitate to contact your authorized supplier, an authorized representative or Ki Mobility. Ki Mobility can be reached at:

(1)715-254-0991

Ki Mobility
5201 Woodward Drive
Stevens Point, WI 54481
U.S.A

www.kimobility.com (Includes Dealer locator and access to printed information)

Or via our Authorized EU Representative:

Etac Supply Center AB
Långgatan 12
33233 Anderstorp
Sweden

Or via our UK Responsible Person:


Etac Ltd
Unit 60, Hartlebury Trading
Estate, Hartlebury
Kidderminster, DY10 4JD

TABLE OF CONTENTS

	Axiom Back Features.....	3
A.	Intended Use.....	4
B.	Your Safety	4
C.	Signal Words.....	4
D.	Symbol Glossary	5
E.	Check Your Skin Daily	6
F.	General Warnings.....	6
G.	Positioning Belts or Harnesses	7
H.	Protecting Your Axiom Back.....	8
I.	Cover Orientation	8
J.	Swing Away Lateral	9
	Set-Up, Adjustment and Care.....	10
A.	Detaching and Reattaching Backrest Assembly.....	10
B.	Pushing or Lifting	13
C.	Axiom Back Transit Safety.....	14
D.	Maintenance	15
E.	Cleaning	16
F.	Sanitizing.....	17
G.	Reassembling Your Axiom Back After Cleaning.....	17
	KI MOBILITY LIMITED WARRANTY.....	18

Axiom Back

Axiom Back Features

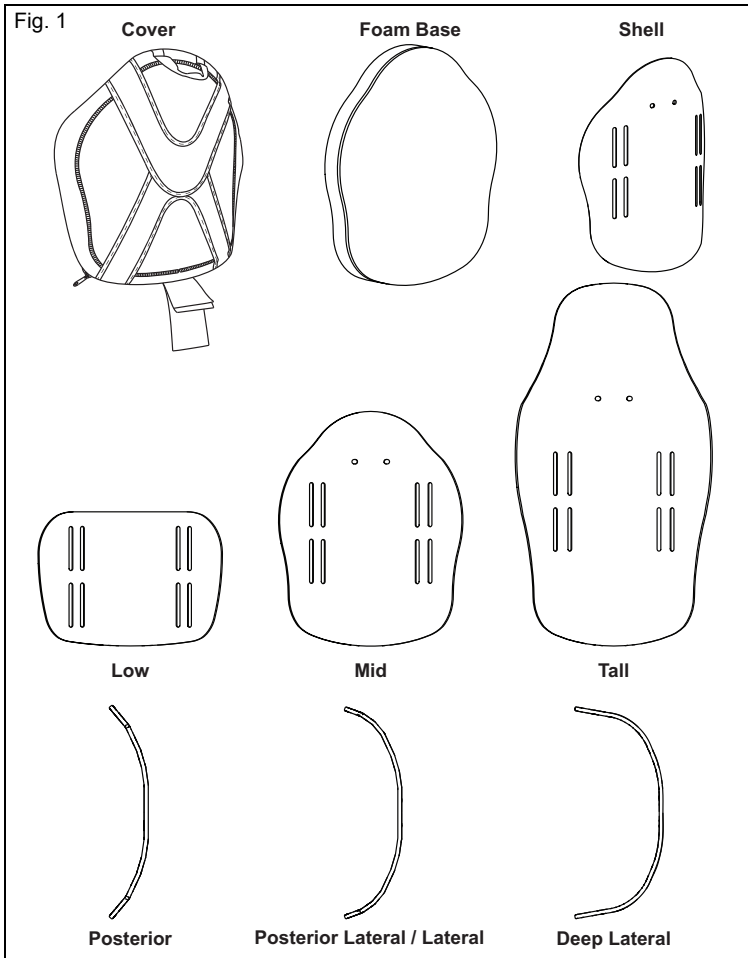MAXIMUM USER WEIGHT: 300 lbs (136 kg)

NOTE: Axiom Custom Back features will vary by order and may not be representative of what is shown in Fig. 1.

A. Intended Use

Ki Mobility Wheelchair Backs are intended to provide postural support, pressure distribution and improved sitting tolerance. This wheelchair back is designed for indoor or outdoor use.

B. Your Safety

Any serious adverse events or injuries related to the use of your product or its accessories must be reported immediately if the incident directly or indirectly leads to serious health decline or death of the user or other person. Report any serious events and/or injuries to Ki Mobility and, if required by local regulation, the competent health authority where the user and/or other person legally reside.

Do not use this product without first reading this entire manual. BEFORE riding, you should be trained in the safe use of this product by an Assistive Technology Professional (ATP) or other competent clinical or technical professional. Ki Mobility manufactures many different products that might meet your needs. The recognized best practice for selecting a product is to consult with an ATP or other competent technical professional and an experienced clinical professional such as a physical therapist, occupational therapist or physician. Final selection of the type of product, options and adjustments rests solely with you and your technical and clinical professionals. The options you choose, and the set-up and adjustment of the product have a direct impact on its performance, stability and its ability to meet your needs.

C. Signal Words

Within this manual you will find what are referred to as “Signal” words. These words are used to identify and convey the severity of varying hazards. Before using this chair you, and each person who may assist you, should read this entire manual. Please note the Signal word and consider any notes, cautions or warnings. Make sure to follow all instructions and use your chair safely. The Signal word refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The “Warnings” are in three main categories, as follows:

NOTE – Note indicates a potentially hazardous situation which, if not avoided, could result in a decline in the performance of the chair or damage to your wheelchair.

CAUTION: Caution indicates a potentially hazardous situation which, if not avoided, could result in damage to your wheelchair and injury to you.

WARNING: Warning indicates a potentially hazardous situation which, if not avoided, could result in serious injury or death.

These signal words will be placed throughout the manual, where appropriate to highlight the hazardous situation. Refer to Section G. for hazardous situations that will apply to the general use of this product.

D. Symbol Glossary

SYMBOL	DEFINITION
	Indicates the medical device manufacturer.
	Date of manufacture (YEAR/MM/DD).
	Indicates the need for the user to consult an instruction for safety reasons such as cautions and warnings. If presented on the medical device or packaging, it indicates the need for the user to consult the instructions for safety reasons such as cautions and warnings.
	Indicates the item is a medical device.
	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Indicates the authorized representative in the European Community.
	Indicates the manufacturer's declaration that the product meets the requirements of the applicable EC directives.
	Indicates the need for the user to consult the instructions for use.
	Indicates the need for the user to consult the listed website for instructions for use in an electronic format.
	Indicates a website where a user may obtain additional information about the medical product.
	Indicates a specified maximum weight limit (lbs/kg).
	Indicates a transit securement point.
	Indicates a potential pinch point.
	Indicates the entity importing the medical device into the locale.
	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Machine Wash, Hot
	Do Not Bleach
	Line dry or tumble, no heat
	Do Not Iron
	Do Not Dry Clean
	Indicates the manufacturer's declaration that the product meets the requirements of the applicable UKCA directives.

E. Check Your Skin Daily

These Axiom Seating and Positioning products are designed to improve pressure distribution and increase sitting tolerance. Whether the cushion provided will meet your individual needs cannot be assured in advance. The integrity of your skin can be affected by many aspects of your daily life and medical condition, including the use of this product. Be sure to follow any skin care regimens established by your clinician.

Users of this product should make sure their skin is inspected daily for changes. If you see changes, discontinue the use of this product immediately and consult with your clinician. A key clinical indicator of tissue breakdown is skin redness. If your skin develops redness, discontinue the use of this product immediately and consult your clinician. Be especially careful to inspect the areas related to the bony prominences of the pelvis such as your trochanters and ischial tuberosities.

WARNING

WARNING: Failure to properly select an Axiom Back that fits your wheelchair can affect your skin integrity which can result in serious injury or death.

WARNING

WARNING: Failure to perform proper skin inspection or continuing to use an Axiom Back after skin changes are present can affect your skin integrity, resulting in serious injury or death.

F. General Warnings

WARNING

WARNING: Do not use or install this Axiom Back without first reading and understanding this manual. Please contact your authorized Ki Mobility supplier or your clinician if you are unable to understand the Warnings and Instructions. Failure to do so could result in serious injury or death.

WARNING

WARNING: The installation of an Axiom Back on a wheelchair could affect the center of gravity of the wheelchair. Changes in your center of gravity may affect your stability in your wheelchair, resulting in tipping over or falling from your wheelchair which may result in serious injury. Always review the instructions for use of your wheelchair to see if changes to the wheelchair may be needed to provide sufficient stability after adding an Axiom Back.

WARNING

WARNING: The integrity of your skin can be affected by many aspects of your daily life and medical condition, including the use of this product. Be sure to follow any skin care regimens established by your clinician. Consumers of this product should make sure their skin is inspected routinely for changes as directed by their clinician. Failure to do so could result in serious injury or death.

G. Positioning Belts or Harnesses

WARNING: Improper use of positioning belts can cause the user to slide underneath the positioning belt in the wheelchair. If this should happen, chest compression or suffocation due to pressure from the belt can occur. If this occurs, the user's breathing may be hampered causing serious injury or death.

WARNING: Improper use of positioning belts could cause body parts to become entangled which can restrict mobility and movement, which could result in serious injury or death.

Ki Mobility recommends the use of positioning belts. Positioning belts can reduce the risk of falling from your wheelchair.

Positioning belts, such as pelvic positioning belts and anterior trunk harnesses and straps, are designed to assist, retain and support proper positioning and posture in the wheelchair.

- Ensure the user does not slide underneath the positioning belt in the wheelchair seat.
- The positioning belt should have a snug fit; tight enough to hold their position, but not so tight as to restrict breathing. An open hand should be able to fit between the belt and the user.
- Proper use of cushions can contribute to pelvic stability and reduce sliding.
- NEVER Use Positioning Belts:
 - a. As a restraint, unless you can remove the belt easily in an emergency, then it may be considered a restraint. A restraint requires a doctor's order, consult with your health care advisor for other options to help you with your posture.
 - b. On a user who is unconscious or agitated.
 - c. As an occupant restraint or safety belt in a motor vehicle. A positioning belt or harness is not designed to replace a vehicle or wheelchair mount occupant restraint. During a sudden stop, with the force of the stop, the user would be thrown forward. Depending on the force this may cause the positioning belt or harness to fail. Reference the Transit Section of this manual for further information.

H. Protecting Your Axiom Back

Avoid prolonged exposure to adverse environmental conditions. This may include excessive exposure to sunlight and temperatures, either hot or cold. This can result in the degradation of the performance of your Axiom Back.

WARNING: Failure to protect your Axiom Back may lead to changes that can affect your skin integrity, resulting in serious injury or death.


Although your Axiom Back has been manufactured in accordance with its labeled flammability standards, you should avoid exposing your cushion to open flames or extreme heat sources such as a burning cigarette. This may cause your cushion to ignite or emit harmful gases.

WARNING: Failure to protect your Axiom Back from extreme heat may cause your Axiom Back to ignite or emit harmful gases, resulting in serious injury or death.

I. Cover Orientation

Your Axiom Back was designed to be seated in a specific orientation and it must be placed properly in your wheelchair with the cover oriented correctly with the foam base. It is labeled to indicate the proper orientation. See Fig. 2.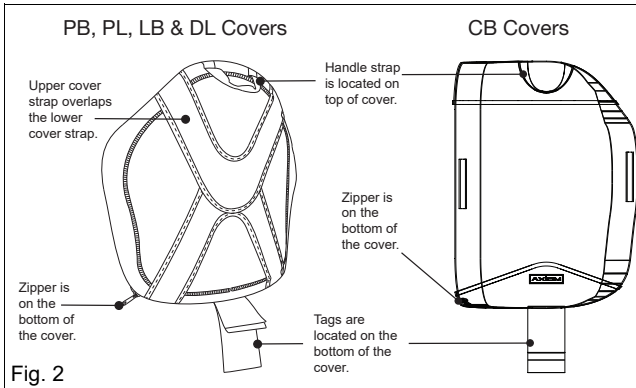


WARNING: Failure to properly orient the Axiom Back cushion and cover can affect your skin integrity, resulting in serious injury or death.

J. Swing Away Lateral

Use

1. Push in the red tab on the swing lateral hinge. Swing the lateral to desired location.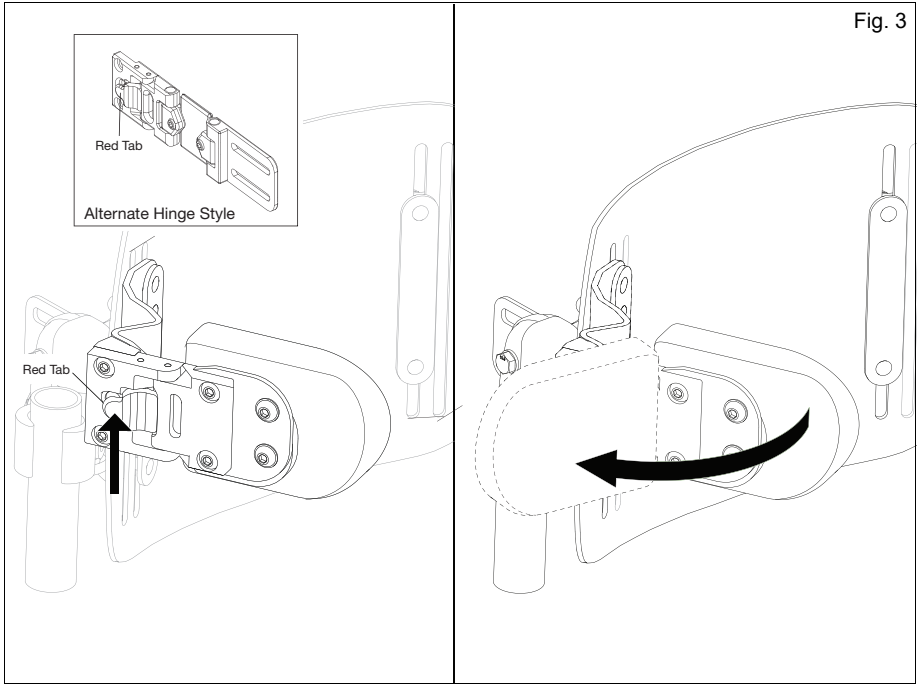


2. Return the lateral back to the default position and ensure it "clicks" or locks into place.

Set-Up, Adjustment and Care


A. Detaching and Reattaching Backrest Assembly

Identify the Hardware Type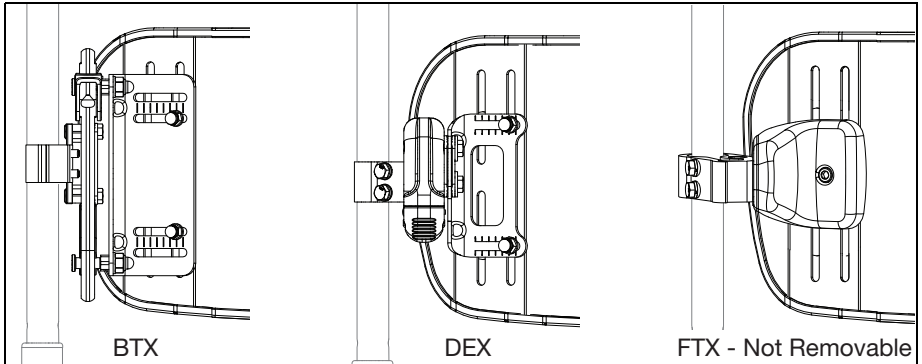


BTX Removal

1. BTX - Remove safety lock pin (Fig. 4:A).

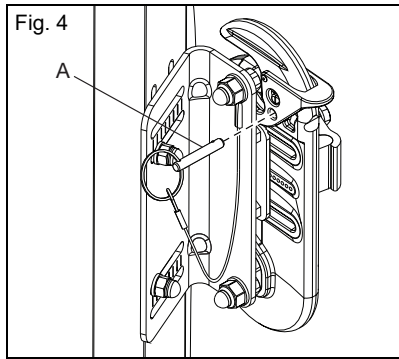
2. Set latch mechanism to the release position (Fig. 5).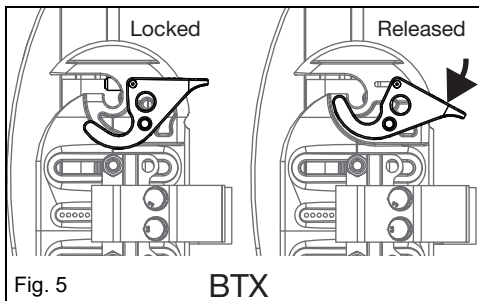


A. Detaching and Reattaching Backrest Assembly (Continued)

3. Pull forward and remove with the lower slot coming out first. See Fig. 6.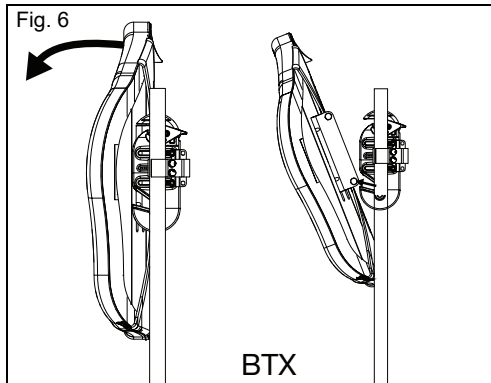


BTX Reinstall

1. Reinstall the back. See Fig. 7.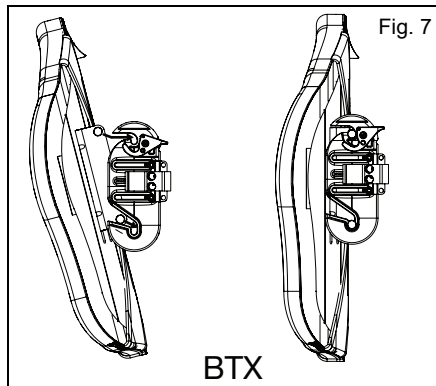

2. Latch Test: Visually verify latch is locked (Fig. 5). Pull up and forward to assure the back is latched. See Fig. 8.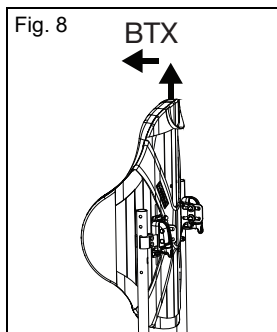


3. Replace safety lock pin (Fig. 4:A).


A. Detaching and Reattaching Backrest Assembly (Continued)

DEX Removal

1. Set latch mechanism to the release position (Fig. 9).

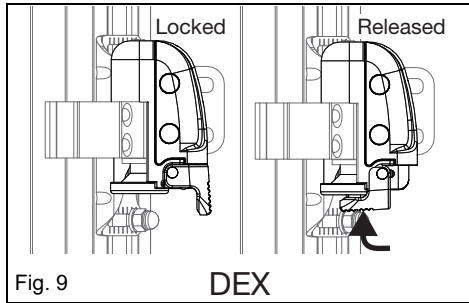
2. Remove the back by lifting up the strap. See Fig. 10.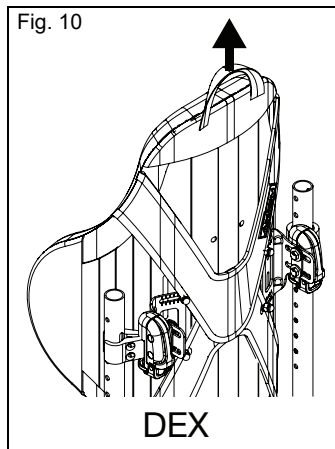


DEX Reinstall

1. Reinstall the back. See Fig. 11.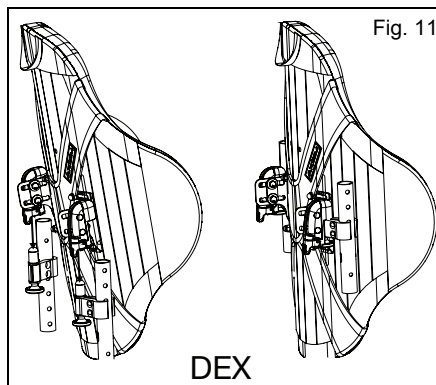

A. Detaching and Reattaching Backrest Assembly (Continued)

2. Latch Test: Visually verify latch is locked (Fig. 9). Pull up to assure the back is latched. See Fig. 12.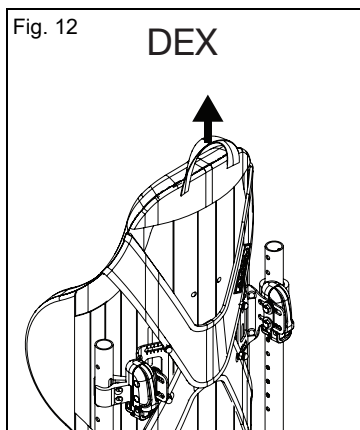

B. Pushing or Lifting

WARNING

WARNING: Do not use the Axiom Back to lift or push the wheelchair.

1. Do not use the Axiom Back handle to lift the weight of the wheelchair. It is intended only for Axiom Back removal.
2. Pushing or pulling is considered improper use of the Axiom Back and may cause the Axiom Back to unexpectedly detach from the wheelchair. Failure to heed these warnings can cause damage to your chair, or cause a fall, tip-over or loss of control may occur and cause severe injury to the user or others.

C. Axiom Back Transit Safety

The Axiom Back has been tested according to the dynamic test requirements of ANSI/RESNA WC-4, Section 20: Wheelchair seating systems for use in motor vehicles or ISO 16840-4 Wheelchair seating - Part 4: Seating systems for use in motor vehicles. A complete system of wheelchair frame, seating, wheelchair tie-down and occupant restraint systems and a properly equipped motor vehicle, that have all complied with the standards mentioned in this section, should be in place before using an Axiom Back in a wheelchair that is used as a seat in a motor vehicle.

WARNING

WARNING: Failure to follow all instructions for installation, use and maintenance within this manual as well as the transit instructions listed below, could result in serious injury or death when using the Axiom Back in a wheelchair that is used as a seat in a motor vehicle.

1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraints.
2. The distance between the top of the user's shoulder and the top of the Axiom Back should not exceed 5.5" (14 cm).
3. The wheelchair must be labeled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
4. If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
5. When using your wheelchair as a seat in a motor vehicle the rider must be in a forward-facing position.
6. The rider must use a Wheelchair Tie-down and Occupant Restraint System that complies with ANSI-RESNA WC-4, Section 18: Wheelchair tie-down and occupant restraint systems for use in motor vehicles or ISO 10542-1 Technical systems and aids for disabled or handicapped persons -- Wheelchair tie-down and occupant-restraint systems -- Part 1: Requirements and test methods for all systems and installed, used and maintained according to the manufacturer's instructions.
7. Use of lap belts, chest straps, shoulder harnesses, any other positioning strap system or positioning accessory should not be used, or relied on as an occupant restraint, unless it is marked as such by the manufacturer in accordance with ANSI-RESNA WC-4, Section 18 or ISO 10542-1, Part 1.
8. Use of headrests, lateral supports or other positioning accessories should not be relied on as an occupant restraint, unless it is marked as such by the manufacturer in accordance with RESNA WC-4, Section 18 or ISO 10542-1, Part 1 or ANSI/RESNA WC-4, Section 20: Wheelchair seating systems for use in motor vehicles or ISO 16840-4 Wheelchair seating -- Part 4: Seating systems for use in motor vehicles.
9. Aftermarket accessories such as trays, oxygen tank holders, oxygen tanks, IV poles, backpacks, pouches and other items should be removed and secured separately in the motor vehicle. In the event of an accident, these items can become dangerous projectiles which may result in serious injury or death for you or other occupants of the motor vehicle.
10. This product is intended for use by adults ranging in weight from 79 to 300 lbs (36 to 136 kg).
11. If the Axiom Back has been involved in an accident, you should not continue to use it, as it may have suffered fatigue that may not be visible.

D. Maintenance

Regular cleaning and maintenance may help extend the life of your Axiom Back. You should inspect your Axiom Back monthly. During cleaning, component inspection is recommended.

1. Check Cover for Tears and Excessive Wear
 - a. Remove cover and check for abnormalities (inside and out).
 - b. Check the foam for breakdown or degradation.
 - a. Tears deeper than .25" (6.35 mm)
 - b. Loose granules of foam.
 - c. Permanent changes to the unloaded foam shape, such as flattening.
2. Check the hardware for any looseness, signs of stripping, degradation or deformities.

WARNING

WARNING: If you find any abnormalities, such as the foam breaking down or damage to the hardware or fasteners, discontinue the use of this product immediately and consult with your Authorized Ki Mobility Supplier. Failure to do so can affect your skin integrity, resulting in serious injury or death.

WARNING

WARNING: The owner of this equipment is responsible for making sure that it has been set up and adjusted by an Authorized Ki Mobility Supplier under the direction of a healthcare professional. It may require periodic safety and function checks or certain in-use adjustments that can be performed by the user, caregiver or Authorized Ki Mobility Supplier.

1. Fasteners: Ki Mobility recommends that all fasteners be inspected for proper function, such as loose bolts or broken components every six months. Loose fasteners should be retightened according to the installation instructions.
2. Replacing Components: Contact your Authorized Ki Mobility Supplier immediately to replace any components. Discontinue use of the system after identifying loose or broken components.

E. Cleaning

1. Axiom Back Cover


NOTE: Drying with heat may cause shrinking that affects the covers fit and function.

- Remove the cover from Axiom Back by detaching the Velcro®. Unzip the zipper on the bottom of the cover.
- Slightly roll and remove the foam to turn the cover inside out.
- Zip the cover closed to avoid snagging.
- Machine wash in hot water (60°C).
- Line dry or tumble, no heat.

NOTE: Do not dry clean covers or use industrial washers or dryer to clean covers. Do not steam autoclave. Do not bleach. Laundry care symbol definitions are shown in Fig. 14.

2. Axiom Back Assembly

- Remove the whole back assembly to make cleaning easier. See Fig. 13.

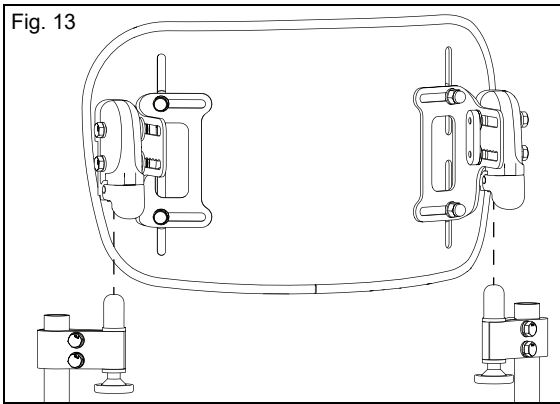- The Axiom Back Shell should be wiped clean with soap and water on a regular basis to prevent buildup of dirt and grime.

3. Positioning Components

- Positioning components can be wiped clean with soap and water.
- DO NOT immerse positioning components underwater.

Fig. 14 Care & Use Instructions

	Machine Wash, Hot
	Do Not Bleach
	Line dry or tumble, no heat
	Do Not Iron
	Do Not Dry Clean


F. Sanitizing

Sanitizing step should be done in addition to regular cleaning:

- a. Remove and launder cover according to steps 1a., 1b., and 1c. above.
- b. Machine wash cover only in hot water (60°C).
- c. Drip and tumble dry on No Heat.
- d. Disinfect all surfaces (cover, foam and back assembly) with over-the-counter disinfecting sanitizer of at least 70% alcohol or wipes. Do not soak or allow pooling of cleaning solutions.
- e. Allow sanitizer to remain on surface for at least 15 minutes and remove with aseptic cloth if necessary.

G. Reassembling Your Axiom Back After Cleaning

1. After Cleaning
 - a. Reinstall foam and cover. Slightly roll the foam when installing cover.
2. Back Assembly Reinstallation
 - a. To reattach the Axiom Back and hardware receivers, line up the pins to the receivers.
 - b. Push the Axiom Back into place on the receivers.
 - c. If properly inserted, the pins will "click" into a locked position.

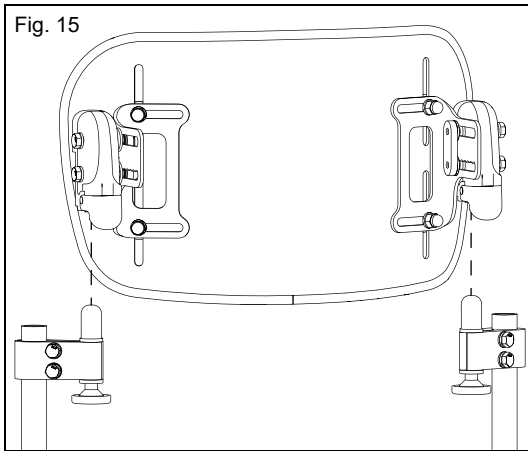WARNING

WARNING: If you are uncertain that you have heard the locking "click", pull up on the Axiom Back using the handle to test and ensure it has been securely locked. If you fail to heed this warning, damage to your equipment, a fall, or loss of control may occur and cause severe injury to the rider or others.

KI MOBILITY LIMITED WARRANTY

The expected life of the product is 5 years.

A. For 24 Months

Ki Mobility warrants the Axiom Back to be free from defects in materials and workmanship for a period of 24 months. This limited warranty applies to:

1. Axiom Back shell and foam

For a period of 24 months (2 years) from the date of purchase, provided normal use, should a defect in materials or workmanship occur, Ki Mobility will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears, or burns.

B. For 6 Months

Ki Mobility warrants the cushion to be free from defects in materials and workmanship for a period of 6 months. This limited warranty applies to:

1. Removable covers

For a period of 6 months from the date of purchase, provided normal use, should a defect in materials or workmanship occur, Ki Mobility will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns.

C. Limitations

1. We do not warrant:
 - a. Damage from neglect, accident, misuse, or from improper installation or repair.
 - b. Products modified without Ki Mobility's express written consent.
 - c. Damage from exceeding the weight limit.
2. This warranty is VOID if the original Axiom Back tag is removed or altered.
3. This warranty applies in the USA only. Check with your supplier to find out if international warranties apply.
4. This warranty is not transferable and only applies to the first consumer purchase of this product through an authorized Ki Mobility supplier.

D. What We Will Do

Our sole liability is to repair or replace covered parts. This is the exclusive remedy for consequential damages.

E. What You Must Do

1. Obtain from us, while this warranty is in effect, prior approval for return or repair of covered parts.
2. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

F. Notice to Consumer

1. If allowed by law, this warranty is in place of any other warranty (written or oral, express or implied, including a warranty of merchantability, or fitness for a particular purpose).
2. This warranty gives you certain legal rights. You may also have other rights that vary from state to state.

Record your lot number (located on cover tags) here for future reference:

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