



StarLock

The Vertical Air Cell Cushion that locks in place



Vertical Air Cells

Star cushions feature vertical air cell technology designed specifically for individuals at high risk of pressure injuries.

Star's interconnected vertical air cell design ensures internal cell pressures equalise as the user sits on the cushion. The vertical air cell construction also effectively reduces surface tension and enables excellent envelopment.



Precision Manufacturing

The Star range is manufactured in Sweden with cutting-edge robot production to ensure precision and consistency.



StarLock – No Compromises

In air cell cushions, the combination of pressure redistributing qualities and stability have always been a difficult compromise – With StarLock, you can have both.

Pressure Care with Pressure Redistribution and Postural Support

StarLock is the complete cushion solution for pressure care with pressure redistribution, stability and postural support. It is available in 3", 4" and 5" (7.5, 10, 13 cm) cell heights.

What ever the cell height, the sitting height will always be the same. However, the higher the cell, the greater the pressure redistribution.

The 5" (13 cm) cell height is unique to StarLock and offers greater immersion and envelopment.

Shaping, Offloading and Stability


The StarLock, with its StarLock Technology, provides the possibility to lock air into each vertical air cell which makes it possible to create a desired shape, offload localized pressure and increase stability, allowing for individualized postural support.

It can be adjusted and reset again and again, making it very flexible over time.

StarLock can also work as a single air chamber cushion allowing individual setup for optimal pressure redistribution.




Features

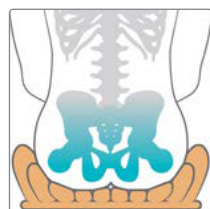


Pressure Redistribution

StarLock is the only air cell cushion with a 5" (13 cm) cell height - that redistributes pressure over a larger surface, provides greater immersion into the cushion, and continually readjusts to the user's shape as they move in their seat.

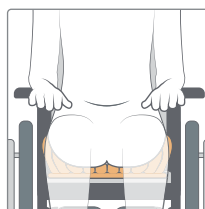


Adjustability

StarLock can be adjusted and reset again and again, making it very flexible over time. The user can remain seated throughout the assessment.

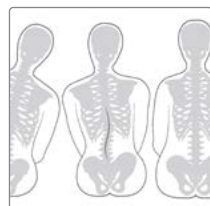


Shape for the Individual

StarLock technology conforms to the user's body shape or desired position, which can help by locking the air into each cell. StarLock can meet the individual user's changing needs for support and shape.

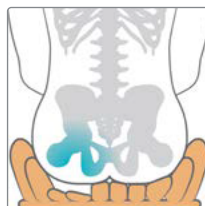

Offloading

Locking down cells in a selected area of the cushion can relieve pressure on particularly vulnerable areas.

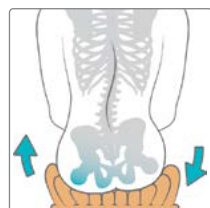


Stability

The individual cell locking technology aids stability and allows individualized postural support.



Recondition and Reuse

StarLock can always be washed and repaired, and it can be adjusted and disinfected as often as you want. Without locking the Lock Pockets, StarLock can also be used as a classic single chamber cushion.

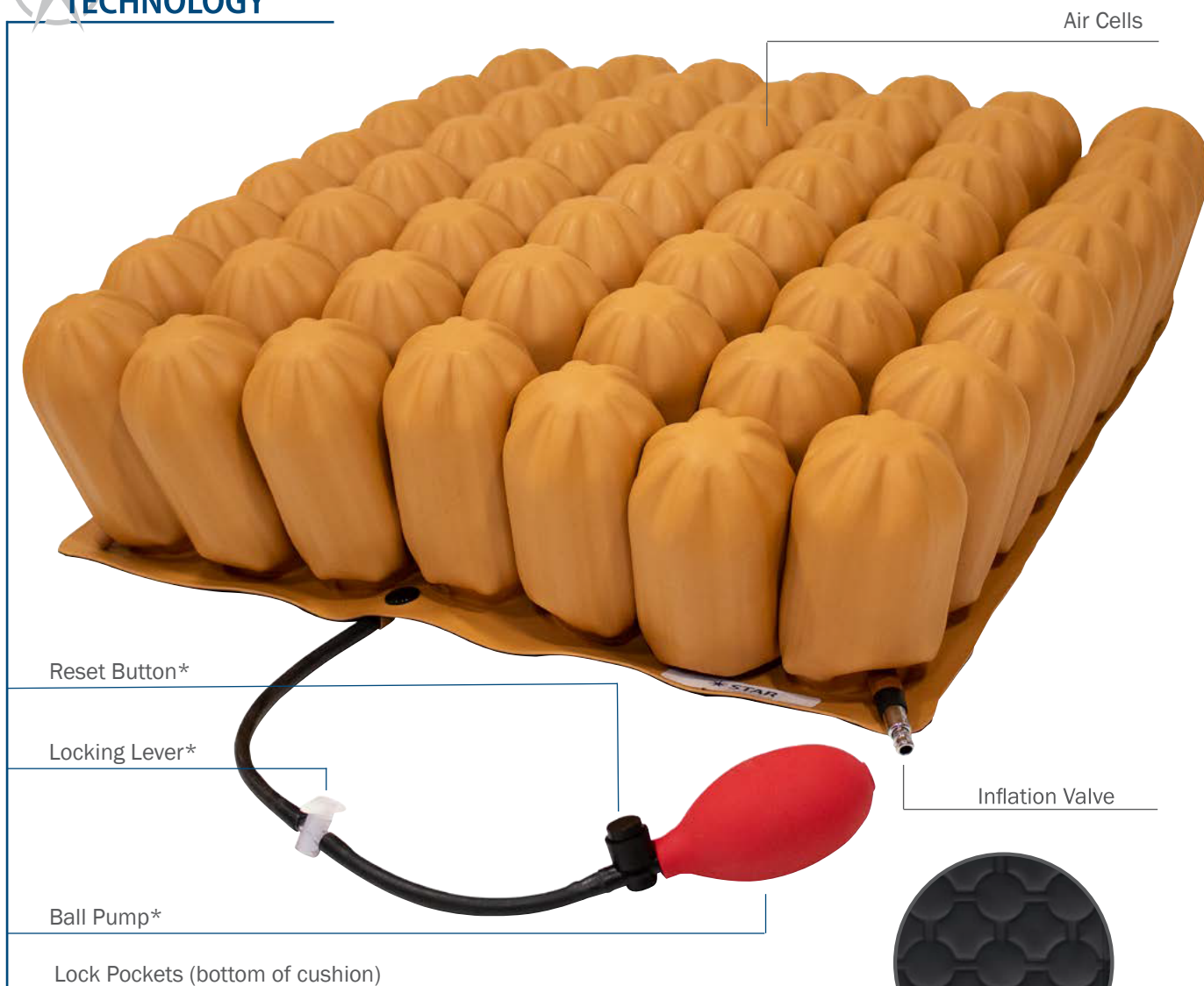

*Reset Button, Locking Lever and Ball Pump activate and deactivate the Lock Pockets.

StarLock cushions come complete with a fitted cover, hand pump, manual and repair kit.



StarLock Size Guide

StarLock WxD	3"	4"	Seat Width*	StarLock WxD	5"	Seat Width*
13.5" x 13.5"	ESSL31313-1	ESSL41313-1	12" - 14"	14" x 14"	ESSL51414-1	13" - 15"
13.5" x 15.5"	ESSL31315-1	ESSL41315-1	12" - 14"	14" x 16"	ESSL51416-1	13" - 15"
13.5" x 17.25"		ESSL41317-1	12" - 14"	14" x 18"	ESSL51418-1	13" - 15"
15.5" x 15.5"	ESSL31515-1	ESSL41515-1	14" - 16"	16" x 16"	ESSL51616-1	15" - 17"
15.5" x 17.25"	ESSL31517-1	ESSL41517-1	14" - 16"	16" x 18"	ESSL51618-1	15" - 17"
15.5" x 19.25"	ESSL31519-1	ESSL41519-1	14" - 16"	16" x 20"	ESSL51620-1	15" - 17"
15.5" x 21"		ESSL41521-1	14" - 16"	16" x 22"	ESSL51622-1	15" - 17"
17.25" x 15.5"	ESSL31715-1	ESSL41715-1	16" - 18"	18" x 16"	ESSL51816-1	17" - 19"
17.25" x 17.25"	ESSL31717-1	ESSL41717-1	16" - 18"	18" x 18"	ESSL51818-1	17" - 19"
17.25" x 19.25"	ESSL31719-1	ESSL41719-1	16" - 18"	18" x 20"	ESSL51820-1	17" - 19"
17.25" x 21"	ESSL31721-1	ESSL41721-1	16" - 18"	18" x 22"	ESSL51822-1	17" - 19"
19.25" x 15.5"	ESSL31915-1	ESSL41915-1	18" - 20"	20" x 16"	ESSL52016-1	19" - 21"
19.25" x 17.25"	ESSL31917-1	ESSL41917-1	18" - 20"	20" x 18"	ESSL52018-1	19" - 21"
19.25" x 19.25"	ESSL31919-1	ESSL41919-1	18" - 20"	20" x 20"	ESSL52020-1	19" - 21"
19.25" x 21"	ESSL31921-1	ESSL41921-1	18" - 20"	20" x 22"	ESSL52022-1	19" - 21"
21" x 17.25"	ESSL32117-1	ESSL42117-1	20" - 22"	22" x 18"	ESSL52218-1	21" - 23"
21" x 19.25"	ESSL32119-1	ESSL42119-1	20" - 22"	22" x 20"	ESSL52220-1	21" - 23"
21" x 21"	ESSL32121-1	ESSL42121-1	20" - 22"	22" x 22"	ESSL52222-1	21" - 23"

* If placed on a wheelchair, the cushion base edges may fold upwards. Hence the difference between the base measurements and recommended seat width.

User Weight:

No limits as long as the correct sized cushion is used for the user.

Cleaning:

Cover: Machine wash in warm water, maximum 60 °C/140 °F. Use a mild detergent. Do not bleach. Rinse in cold water and tumble dry on low temperature or line dry.

Cushion: Close the inflation valve. Clean the cushion by hand with soap, bicarbonate or vinegar. For disinfection, use a 70% alcohol-based disinfectant solution.

Covers



Standard Cover

Cover is breathable and flame resistant polyester with 4-way stretch.



Incontinence Cover

Cover is breathable and flame resistant nylon with 4-way stretch and concealed zipper.

A hand is shown inflating a black inflatable device with a red bulb. The device has a repeating pattern of circular pockets. A white circle with an arrow points to one of these pockets. The device is connected to a black hose and a red bulb. The background is white.

Lock Pockets – the secret to StarLock

Setting up StarLock

StarLock can easily be set up for the individual user's needs. Once the air level is set, the air can then be locked into the individual cells to create the correct support.



Quick Guide StarLock

Click or Scan the QR code to watch the quick guide video for setting the air level for StarLock.



StarLock Product Page

Click or Scan the QR code to find more information about StarLock on StarLock product page, on kimobility.com





Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the products' intended purpose and latest information, please visit www.etac.com